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SUCTION AND IRRIGATION KIT Reference: AS12/201B The following instructions must be read before using the product.

INSTRUCTIONS FOR USE Indications: The suction/irrigation kit is indicated for use in laparoscopy surgery to suction residual fluids and for irrigation during a procedure.

- Symbols used: See instructions for use, Sterilized with Ethylene Oxide Expiry date: YYYY-MM, Date of manufacture: YYYY-MM, Manufacturer, Do not use if packaging is damaged, Reference, Protect from moisture, See the precautions for use, Batch N°, Single use, Do not re-sterilize, Approved European Representative in the European Community, Cardboard box, Latex free, Keep out of sunlight, Temperature limits: 15°C - 25°C

Contra-indications: The suction/irrigation kit is not indicated when laparoscopy techniques are contra-indicated.

Instrument description: The suction/irrigation kit is a sterile, multi-function, single use instrument for irrigation and suction. This kit is available with a 5 or 10 mm diameter cannula and with a double perforator.

Instructions for use: Remove the kit from its packaging. Before removing the protector from the perforator and inserting it into the bag (or pump), please clamp the tube.

Precautions: This instrument is designed for single use only and must not be re-used or re-sterilized. Re-using or re-sterilizing this device may cause it to fail and result in injury to the patient.

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If you would like further information, please contact: Distributed/Distribué par:

Peters SURGICAL Immeuble AURELLEUM - 1 cours de l'Ile Seguin 92100 BOULOGNE-BILLANCOURT FRANCE

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